

# 5008S

## Service Manual

Edition: 9/01.11  
Part no.: M46 081 1



Fresenius Medical Care



# Table of contents

## 1 Index

## 2 Important information

2.1	Important information on the Service Manual .....	2-1
2.2	Important information on the device .....	2-2
2.3	Addresses .....	2-3

## 3 Installation

3.1	Important information on the installation / initial start-up.....	3-1
3.2	Initial start-up report .....	3-1
3.3	Explanations on the initial start-up report .....	3-7

## 4 Specifications

4.1	Dimensions and weight .....	4-1
4.2	Type label (device identification) .....	4-1
4.3	Electrical safety (classification according to EN 60601-1, IEC 601-1) .....	4-2
4.4	Electrical supply .....	4-2
4.5	Fuses .....	4-2
4.6	Guidance and manufacturer's declaration on EMC (IEC 60601-1-2:2001) .....	4-3
4.7	Operating conditions .....	4-5
4.8	External connection options .....	4-6
4.9	Override conditions.....	4-6
4.10	Operating programs .....	4-7
4.11	Dialysate circuit and safety systems .....	4-8
4.12	Extracorporeal blood circuit and safety systems .....	4-11
4.13	DIASAFE <sup>®</sup> plus .....	4-13
4.14	OCM .....	4-13
4.15	ONLINEplus <sup>™</sup> .....	4-13
4.16	Single-Needle (option) .....	4-14
4.17	BPM (option) .....	4-14
4.18	BTM (option) .....	4-15

4.19	Network.....	4-16
<b>5</b>	<b>Setup</b>	
5.1	SYSTEM SCREEN settings .....	5-1
5.2	Operator setup .....	5-2
5.3	Technician's setup .....	5-29
<b>6</b>	<b>TSC / maintenance</b>	
6.1	Important information regarding the procedure.....	6-1
6.2	Accessories and supplies.....	6-2
6.3	TSC / MA test report .....	6-2
6.4	Explanations on the TSC / MA test report .....	6-7
6.5	TSC report .....	6-31
<b>7</b>	<b>Error messages</b>	
<b>8</b>	<b>Tools (service equipment)</b>	
<b>9</b>	<b>Calibration / adjustment</b>	
9.1	Selecting the Service menu .....	9-1
9.2	Service menu, contents .....	9-1
9.3	Flow diagram.....	9-1
9.4	Calibrate .....	9-2
9.5	Diagnostics .....	9-9
9.6	Setup.....	9-12
9.7	Module .....	9-12
9.8	Various.....	9-12
9.9	Update.....	9-12
<b>10</b>	<b>Repair</b>	
10.1	Precautions for working on the hemodialysis system.....	10-1
10.2	Component overview .....	10-2

10.3	Assembly of components .....	10-10
10.4	Equipment .....	10-12
10.5	Placing the system on its side / carrying the system .....	10-13
10.6	IV pole .....	10-14
10.7	Housing .....	10-15
10.8	Monitor .....	10-19
10.9	Updating the monitor operating system M1 .....	10-26
10.10	Power supply unit and power cable .....	10-27
10.11	Extracorporeal Blood Circuit Module EBM .....	10-28
10.12	Hydraulics .....	10-48
10.13	Cart, braking mechanism and concentrate container holder.....	10-66
10.14	Removing and installing the battery.....	10-69
10.15	Checking the network interface .....	10-70
10.16	Printed circuit boards .....	10-72

## 11 Functional description

11.1	Complete system.....	11-1
11.2	P.C.B.s, overview.....	11-3
11.3	Monitor .....	11-4
11.4	EBM (Extracorporeal Blood Circuit Module).....	11-6
11.5	Hydraulics (block diagram / description).....	11-8
11.6	Power supply unit.....	11-10
11.7	Pneumatics .....	11-12
11.8	Hydraulics (flow diagram / legend / description).....	11-14
11.9	Endless rinse program.....	11-18

## 12 Service program (option)

It's a preview. You can download the full file by clicking the link below.

<https://shopservicemanual.com/>

Service Manuals from 2\$